



# Chicken testicle collection for virology and histopathology


Frederic J. Hoerr, DVM, PhD

September 4, 2021

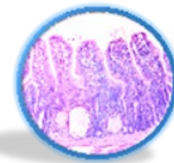

**Veterinary Diagnostic  
Pathology, LLC**

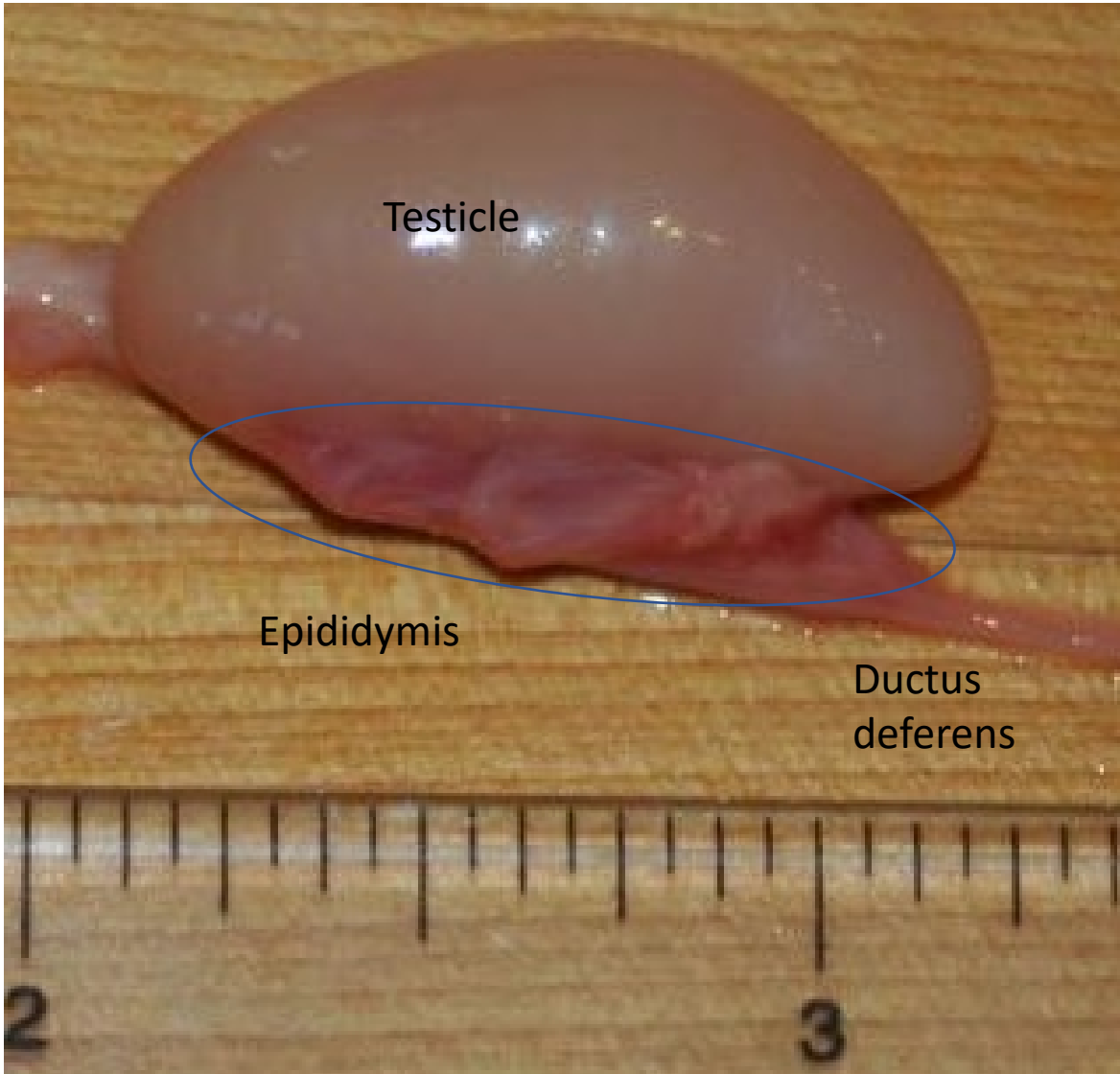

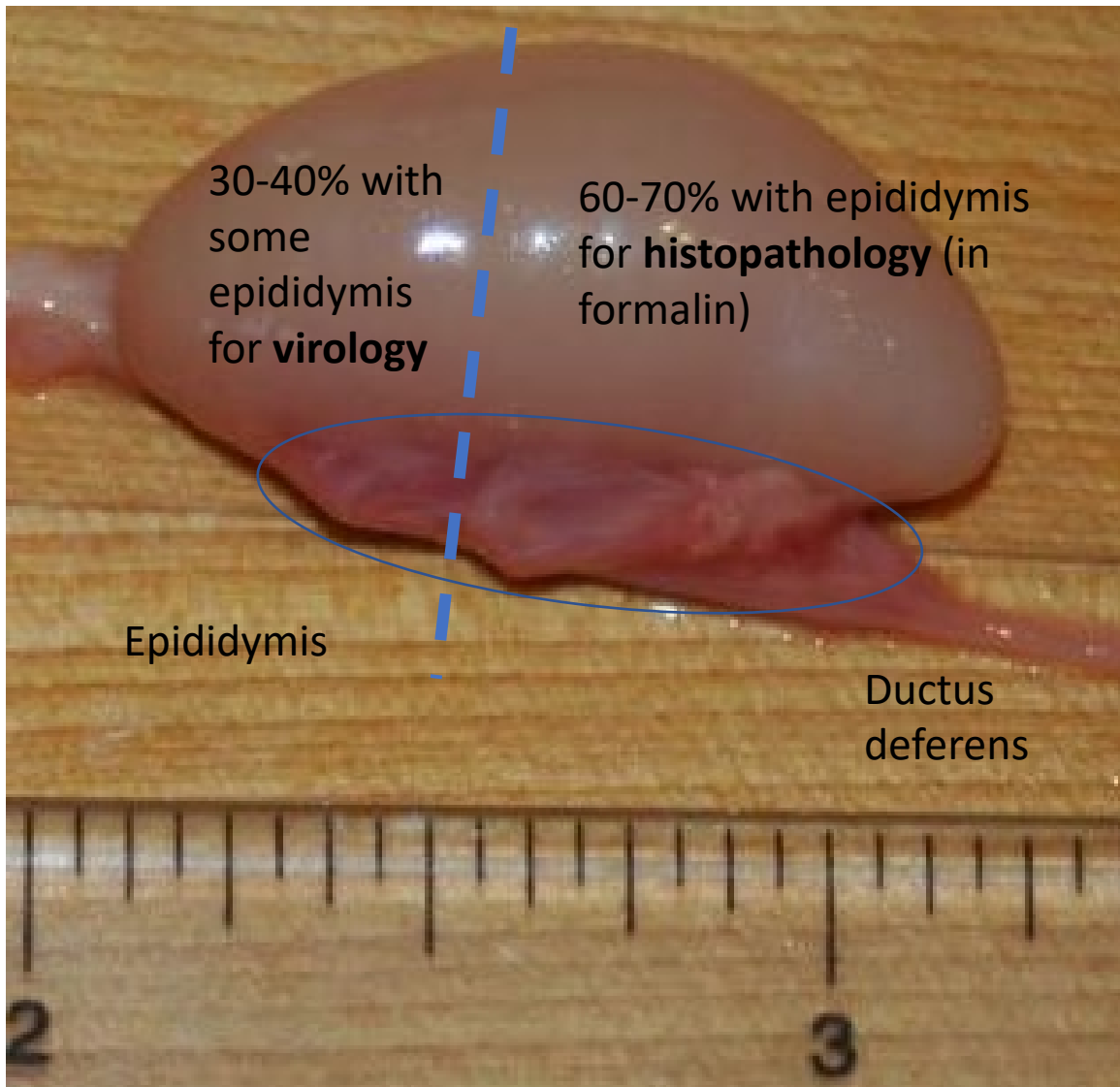

- Be gentle with the tissue
- Do not squeeze
- Cut with a sharp knife or scalpel (prefer not with scissors, which compresses the tissue before cutting)

- Virology specimen to whirl Pak or sturdy small Ziplock bag, freeze or store per instructions from virology lab.
- Histopathology specimen to surgical specimen formalin (regular formalin sample jar).
- Both testicles can go in one jar provided there is at least 5:1 formalin:tissue volume (preferably 10:1)

END

[fred.hoerr@gmail.com](mailto:fred.hoerr@gmail.com)

[www.vetdx.com](http://www.vetdx.com)

334-750-7566

638 South Fort Valley Rd, Fort Valley, VA 22652



**Veterinary Diagnostic  
Pathology, LLC**